

Trust the science.

InterSpace[®]

Shoulder Spacer



A two-stage exchange with the use of Exactech InterSpace has shown strong eradication rates.¹

19

Peer-reviewed papers at 18 different sites

732

Cases reported

94.1

Percent eradication rate at implant removal

93.8

Percent eradication rate at latest follow-up

55

Months average follow-up

Surgeon focused. Patient driven.[™] **Exactech[®]**

InterSpace® Shoulder Spacer

The InterSpace® Shoulder temporarily replaces a Total Shoulder Arthroplasty (TSA) that has been explanted due to infection. InterSpace Shoulder is a preformed, partial load-bearing device consisting of Gentamicin-impregnated PMMA bone cement.

ADVANTAGES OF PREFORMED SPACERS

- Proven high release formulation and design yields predictable and consistent local antimicrobial activity compared to other treatment options²⁻⁴
- Preformed spacers are designed to maintain joint space and allows limited mobility with partial weight-bearing^{5-7*}
- Stabilizes or tensions the soft tissues and reduces bone loss between stages, potentially facilitating easier re-implantation during a second-stage procedure^{6,8-9}
- Reinforced with a stainless steel core to provide a more robust mechanical structure
- Improves quality of life between procedures^{6,8}
- Provides predictable, consistent local antibiotic release^{2,3}
- InterSpace demonstrates functional success rates equivalent to non-infected revisions.⁹
- Preformed spacers have been shown to shorten operating room time¹⁰
- Reduces hospitalization and allows for an early transition to rehabilitation and physical therapy^{11,12}


*Partial weight bearing must be assessed on an individual basis with relation to the anatomic condition of the local bone, bone quality and clinical conditions of the patient during rehabilitation stages. Care must be taken to minimize the risk of damaging bone tissue and the implant through excessive weight bearing or forced mobilization.

REFERENCES

1. Data on file at Exactech.
2. **Mutimer J, et al.** Measurements of in vivo intra-articular gentamicin levels from antibiotic loaded articulating spacers in revision total knee replacement. *Knee*. 2009 Jan;16(1):39-41.
3. **Bertazzoni Minelli E, et al.** Antimicrobial activity of gentamicin and vancomycin combination in joint fluids after antibiotic-loaded cement spacer implantation in two-stage revision surgery. *J Chemother*. 2015 Feb;27(1):17-24.
4. **Pattyn C, et al.** Preformed gentamicin spacers in two-stage revision hip arthroplasty: functional results and complications. *Int Orthop*. 2011 Oct;35(10):1471-6. doi: 10.1007/s00264-010-1172-8. Epub 2010 Nov 30.
5. **Romanò CL, et al.** Preformed antibiotic-loaded cement spacers for two-stage revision of infected total hip arthroplasty. Long-term results. *Hip Int*. 2012 Jul-Aug;22 Suppl 8:S46-53.
6. **Castelli CC, et al.** Two-stage treatment of infected total knee arthroplasty: two to thirteen year experience using an articulating preformed spacer. *Int Orthop*. 2014 Feb;38(2):405-12.
7. **Coffey MJ, et al.** Treatment of glenohumeral sepsis with a commercially produced antibiotic-impregnated cement spacer. *J Shoulder Elbow Surg*. 2010 Sep;19(6):868-73.
8. **Burnett S, et al.** Technique and timing of two-stage exchange for infection in TKA. *Clin Orthop Relat Res*. 2007 Nov;464:164-78.
9. **Romanò CL, et al.** Aseptic versus septic total hip arthroplasty revision: comparing the results. Poster presented at the 76th Annual American Academy of Orthopaedic Surgeons Meeting; 2009 Feb 25-28; Las Vegas, NV.
10. **Castelli C, et al.** Key points in two-stage revision for infected knee arthroplasty: bone loss, quality of life between stages and surgical approach at second stage. In: Meani E, et al [ed]. *Infection and local treatment in orthopedic surgery*. Berlin: Springer-Verlag; 2007. 294-301.
11. **Magnan B, et al.** Two-stage revision of infected total hip replacement using a preformed, antibiotic-loaded acrylic cement spacer. In: Meani E, et al [ed]. *Infection and local treatment in orthopedic surgery*. Berlin: Springer-Verlag; 2007. 205-13.
12. **Vecchini E, et al.** Antibiotic-Loaded Spacer for Two-Stage Revision of Infected Total Knee Arthroplasty. *J Knee Surg*. 2017 Mar;30(3):231-237.

InterSpace® is produced by Tecres® S.p.A., Italy, and is distributed only in North America by Exactech, Inc.

Exactech, Inc. is proud to have offices and distributors around the globe. For more information about Exactech products available in your country, please visit www.exac.com.

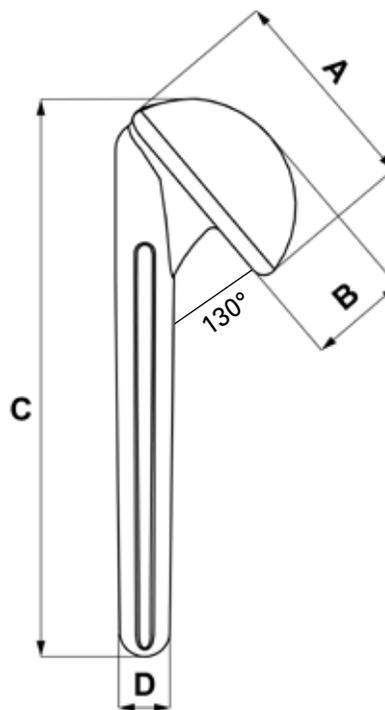

SIZE CHART

Size	REF	A (mm)	B (mm)	C (mm)	D (mm)	Gentamicin Base
Small (S)	SPS0121K	41	16	99	7	0.4g
Large (L)	SPS0021K	46	22	125	11	0.8g

Trials

SPS90Z1 InterSpace Shoulder Trials (S, L)